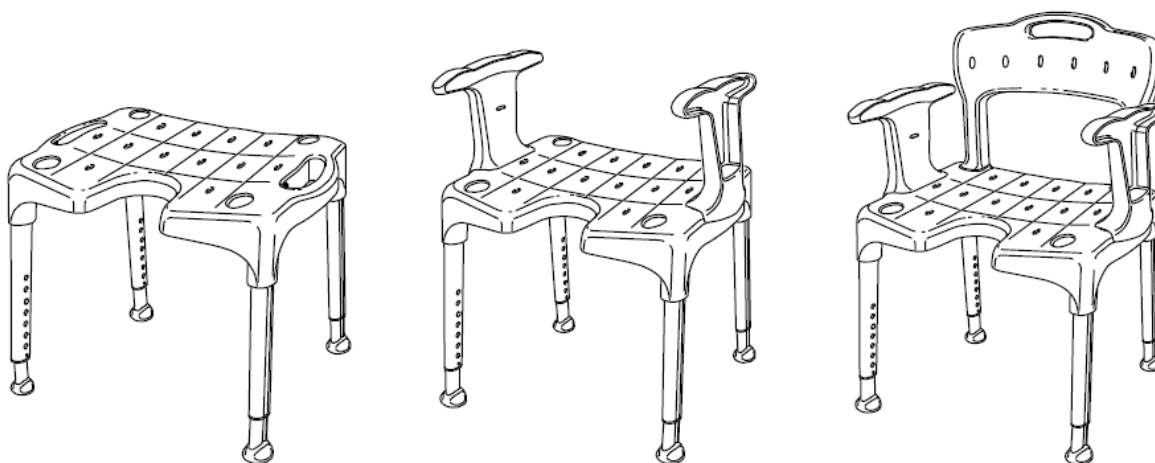
Etac

etac®


Swift / Smart / Easy / Edge

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2015-02-02

Etac Swift

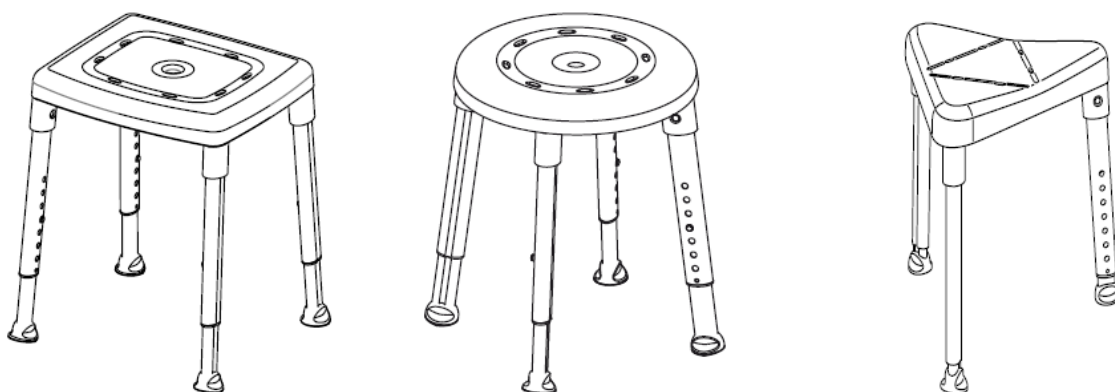Etac Smart

Etac Easy

Etac Edge

CE

VÝROBCE

Etac Supply Center AB
Langgatan 12
Box 203, 334 24 Anderstorp Sweden
tel +46 371 58 73 00
fax +46 371 58 73 90
www.etac.com

DISTRIBUCE A SERVIS

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en

English

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

The images which are referred to on each page are found at the start of the manual (pages 2 to 3). See Figures A to F for the corresponding text.

Intended use

Facilitates sitting when carrying out intimate hygiene.

Description

Our stools/chairs are designed for use in showers or in bathrooms. The products have a stable design with built-in flexibility which adapt to uneven floors. All products are fitted with height-adjustable legs that are simple to attach. The leg ferrules offer good grip and the anti-slip pattern of the seat offer the important safety and stability properties that make the user feel safe. Etac Swift can be anything from a simple shower stool to a complete shower chair, depending on how you choose to equip it.

Swift, Smart, Edge and Easy are tested and comply with the requirements set out in EN ISO 10993-1, EN 12182.

This product should be recycled as per national regulations.

Product Description

See Figure A

A1. Seat A2. Legs A3. Ferrule

A4. Product label

A5. Armrest A6. Backrest

Swift	Smart	Easy	Edge
●	●	●	●
●	●	●	●
●			

Technical data

See Figure B

Assembly

See Figure C

Dismantling

See Figure D

Guarantee

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com

Safety & Maintenance

See Figure E

- A defective product or accessory must not be used.
- E1. A height difference of more than one hole is not allowed for chair legs.
E2. When moving sideways, do not lean on just one armrest.

E3. Max user weight

E4. Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

E5. Can be decontaminated at max. 85°C/185°F for 3 minutes if required.

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Accessories	See Figure F	Swift	Smart	Easy	Edge
F1.	Back support	●			
F2.	Armrest/pair	●			
F3.	Seat pad	●			
F4.	Backrest pad	●			
F5.	Soap dish	●			
F6.	Seat pad		●		
F7.	Seat pad			●	
F8.	Seat pad				●
F9.	Swivel plate	●	●		